



# QUICKIE DUAL FOOTPLATE DRILL GUIDE

Copyright ©2021 LUCI,  
All Rights Reserved.

P002-050 Version 1.0

All product names, brands,  
and trademarks are property  
of their respective owners. All  
company, product, and service  
names used in this website are  
for identification purposes only.  
Use of these names, trademarks,  
and brands does not imply  
endorsement.

For more information visit  
[luci.com/patents](http://luci.com/patents).

Tear this drill guide out along  
the perforated lines. Line up the  
appropriate green slot with the  
footplate as shown  
in the images  
below. Securely  
tape this template  
to the footplate.  
Ensure that it does  
not shift while the  
holes are drilled.

Standard Footplate

Pre-2019 Footplate



Use 5/16 inch drillbit  
for these two holes



Standard Footplate



Pre-2019 Footplate

