

REPLACING WHEELCHAIR BATTERIES WITH LUCI INSTALLED

These instructions are for **V2 LUCI Model 109 units**. You will know these are the correct instructions to follow if:

- LUCI is on a Quickie Q300 M power wheelchair
- The LUCI unit was shipped in October 2023 or after
- The serial number on LUCI is 000144 or greater
- The rear SmartFrame is attached to the side SmartFrames with three screws on each side

If you have any questions, please call our tech support line, 615-813-5824 x2.

To replace the batteries on a Quickie Q300 M that has a V2 LUCI installed, follow these steps:

1. On both sides of the wheelchair, use a 3mm Allen wrench to remove the M5 x 8mm flange head screws (three on each side) and set the LUCI SmartFrame down onto the caster arms.
2. Remove the thumbscrews attaching the seating assembly to the front seat posts and carefully tilt the seating assembly back to gain access to the base of the chair.

⚠ Caution: Use the kickstand and do not go past the kickstand travel to avoid damaging the Rear SmartFrame.

3. Remove the plastic shroud and swap the batteries as you normally would.
4. Close the seating assembly and reinsert the thumbscrews. Clip the Rear SmartFrame into place on the Side SmartFrames, lining up the three standoffs on each side. Re-insert one M5 x 8mm flange screw on each side to hold the Rear SmartFrame in place. Then insert the remaining four M5 x 8mm flange screws and tighten all six with a 3mm Allen wrench. Ensure all cables remain neatly routed to avoid pinching or dragging.

