

REPLACING WHEELCHAIR BATTERIES WITH LUCI INSTALLED

These instructions are for **V1 LUCI Model 109 units**. You will know these are the correct instructions to follow if:

- LUCI is on a Quickie Q300 M power wheelchair
- The LUCI unit was shipped before October 2023
- The serial number on LUCI is 000143 or lower
- The front SmartFrames are attached to the seat rails, not the chair base

If you have any questions, please call our tech support line, 615-813-5824 x2.

To replace the batteries on a Quickie Q300 M that has a V1 LUCI installed, follow these steps:

1. Remove the thumbscrews attaching the seating assembly to the front seat posts.
2. On both sides of the wheelchair, loosen the forward-most and remove the rear-most thumbscrews attaching the rear sensor pods to the LUCI SmartFrame to allow the pods to swing out of the way.
3. Tilt the seating assembly back to gain access to the base of the chair.
▲ Caution: Use the kickstand and do not go past the kickstand travel to avoid damaging the Rear SmartFrame.
4. Swap the batteries as you normally would.
5. Close the seating assembly and reinsert/tight the thumbscrews for the rear LUCI pods as well as the seating assembly. Ensure all cables remain neatly routed to avoid pinching or dragging.

