



Q300 M PEDIATRIC SCOUT INSTALLATION

YOU WILL NEED

- Alcohol Wipe
- Adhesive Mounting Clips
- Scout on Pediatric Scout Bracket


1 Prepare and Test Fit

Verify that there are no mounts, brackets, or other objects that would prevent the mount from sitting flush to the base of the chair in the front and underneath the front part of the base. Test fit the bracket to ensure it fits snugly to the base. Prep the areas where the adhesive mounting tape will contact the base with an alcohol prep wipe.



2 Attach Scout Bracket

Remove the backing from the adhesive mounting tape. Mount the Pediatric Scout Bracket by moving it up to and toward the base simultaneously at a 45-degree angle. Apply pressure to all adhesive mounting tape locations for 30 seconds.

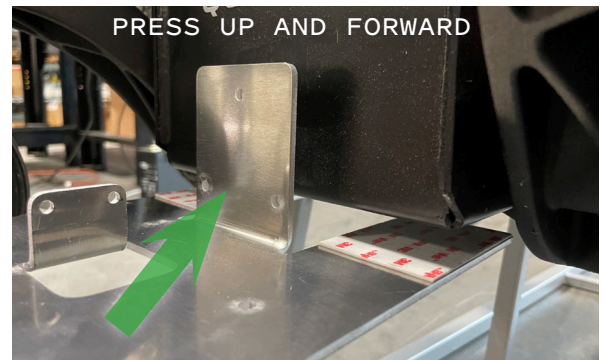

3 Route Scout Cable

Use two adhesive mounting clips to hold the Scout cable to the base. Then, follow the wire routing instructions in the installation guide.

